



*Helping To Create Heart Safe Communities*

## SOS•CPR ♥ Training and AED Programs

101 Golf Course Drive • Suite C-1 • Rohnert Park, CA 94928  
707-795-4444 • Fax 707-585-3705 • [www.soscpr.com](http://www.soscpr.com) • [aed@soscpr.com](mailto:aed@soscpr.com)

### REPLACING THE PHILLIPS HEARTSTART AED PADS

1. Release the existing pad cartridge by sliding the blue foot at the top of the AED device
2. Pop out existing Cartridge and insert Replacement
3. The AED may cycle thru a series of prompts, just follow any directions if given.
4. **DO NOT PULL GREEN HANDLE.** This is only used in an emergency when AED is deployed and PADS need to be used. Pulling the GREEN Handle will expose the PADS to air and dry out the adhesive, requiring PADS to be replaced sooner.
5. Recycle or discard expired or used Pad Cartridge.
6. Record the new expiration date for tracking.



For complimentary tracking of the battery and Pad Expiration of your AED, contact SOS•CPR at (707) 795-444. EMMA (Equipment Maintenance and Monitoring Application) sends you monthly reminders and electronically keeps a record of all your required monthly maintenance checks.